

# PleuraFlow Active Clearance Technology

## Training Guide - New Design

### Product Description

The Shuttle Guide includes the Magnet Strength Button that, when depressed, enables greater magnetic force when needed.



### General Changes

Current Design



New Design



## 24/7 Clinical Support


1-844-257-3569

Access our new clinical support tool, **ClearFlow Assistant**, on the web or download the app by visiting [www.clearflow.com/assistant](http://www.clearflow.com/assistant) or scanning the QR code below.



## Instructions for the OR (Operating Room) Staff

- 1 Upon removing the device from the package, engage the Shuttle Guide with the internal magnet by withdrawing the Shuttle Guide distally against the drain barb.

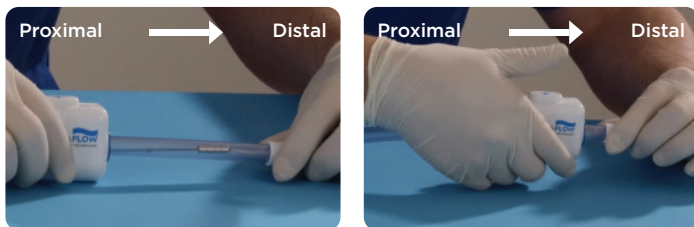

- 2 New Piggyback Label located on outer pouch

3 piggyback sticker labels with UDI bar code provided for use if desired.



- 3 During actuation, if additional magnetic strength is needed, depress and hold the Shuttle Guide Magnet Strength Button.

- Magnet Strength Button is not required to perform an actuation.
- The Magnet Strength Button can be utilized when advancing or retracting the Shuttle Guide.



## Instructions for the ICU/Step-Down Unit Staff

During actuation, if additional magnetic strength is needed, depress and hold the Shuttle Guide Magnet Strength Button.

- Magnet Strength Button is not required to perform an actuation.
- The Magnet Strength Button can be utilized when advancing or retracting the Shuttle Guide.
- The Magnet Strength button may be utilized when decoupling occurs or in situations when the Zip was helpful previously.

